

## Product data sheet




### Information

|             |                                             |
|-------------|---------------------------------------------|
| Designation | Toilets multifunctional grab bar to support |
| Reference   | 048440                                      |

### Product advantages

- + Adaptable to both left or right side.
- + Built-in accessories : no extra drilling. Ideal for small spaces.
- + Easy setting : pre-drilled bars.

### Description

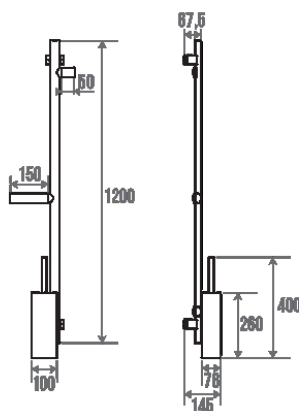

Elliptical aluminium tube 38 x25 mm, polymer fixing covers.

Concealed fixings.

Available for the toilet area, with a toilet roll holder, a toilet brush and holder or a robe hook so as to hang small objects or clothes up : cane, hand bag, hat, scarf, and for the shower area, with a bracket for shower handset and a shelf.

Pre-drilled tubes every 50 mm in order to set the accessories according to the desired height and side : right or left version (reversible).

### Technical characteristics

|                        |                      |
|------------------------|----------------------|
| Dimensions             | 257 x 1226 x 67.5 mm |
| Diameter Ø             | 38 x 25 mm           |
| Material               | Aluminum             |
| Gross weight           | 1.790 kg             |
| Packaging              | Cardboard box        |
| CE Marking : safe load | 100 kg               |
| Laboratory tests       | 225 kg               |
| Made in France         | yes                  |
| Maintenance            | yes                  |
| Guarantee              | 10 years             |
| Net weight             | 1.330 kg             |

|                 |                                  |
|-----------------|----------------------------------|
| Gencod          | 3563390484401                    |
| Custom code     | 7615200000                       |
| Technical sheet | <a href="#">A04841001_IND_04</a> |

## Fixing and maintenance advices

---

Recommended height for the horizontal part : 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

Fiche produit mise à jour le : 07/05/2024