

## Product data sheet





#### Information


Designation White foldaway shower seat

047630 Reference

# Product advantages

- + Maximum safety: with automatic stand.
- + Compact in raised position.
- + Easy fixing
- + Corrosion resistant warranty.

### Description

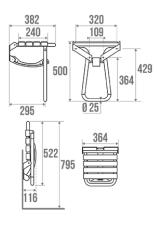

Standard size.

4 white polypropylene slats.

White epoxy-coated aluminium tube.

Enables users to sit down safely and comfortably while showering.

Standard height 500 mm.

#### Technical characteristics

Finish

Color

**Dimensions** 

Diameter Ø

Material

Gross weight Packaging

CE Marking : safe load

Laboratory tests

Epoxy-coated

White


380 x 355 x 500 mm

25 mm

Aluminium & Polypropylene

2.600 kg Quadri box 150 kg 225 kg

Page 1/2

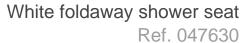



Made in France yes
Maintenance yes
Guarantee 10 years
Net weight 2.174 kg
Gencod 3563390476307

 Custom code
 7615200000

 Technical sheet
 A13470022 IND 21

Spare parts Ref. 047636: pads for shower seat. Ref. 047638: White seat slat. Ref. 047639: Support slat.

## Fixing and maintenance advices

Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. The shower seat provides support and added safety if properly fitted in accordance with our technical instructions. For use in seated position. To guarantee the user safety, DO NOT rise up on the seat. When the seat is raised, DO NOT use it as a grab bar. To avoid any risks of pinching out, DO NOT put your fingers in the mobile parts. The product does not contains any dangerous substance for the health. We disclaim all liability for any personal injury resulting from inappropriate installation or use of this seat.

Fiche produit mise à jour le : 11/06/2024