## Technische fiche

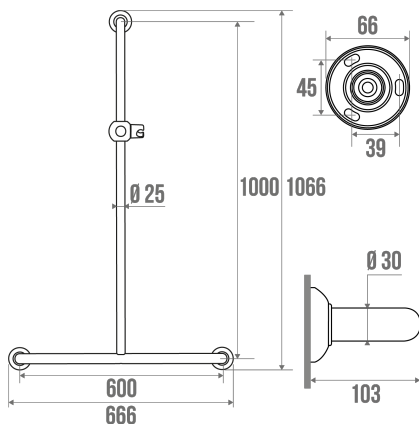
### Informaties

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### De pluspunten

- + Reducing bacteria up to 99 %
- + Improving products hygiene.
- + Minimizing cross-contamination

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Antibacterial treatment reducing bacteria up to 99 %.  
 Concealed fixings.  
 White epoxy-coated aluminium tube Ø 30 mm.  
 White polymeer flanges Ø 66 mm.  
 Supplied with universal clip-on shower rail slider bracket in white ABS.  
 Perfectly suitable with the Durofort antibacterial accessories.

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---

Recommended height for the horizontal part : 800 mm from the ground. Grab bars provide support and added safety if properly fitted in accordance with our technical instructions. Careful consideration and selection of the appropriate wall fixings according to the wall-mounted support is required to ensure the long life of your accessory. The use of stainless steel screws (Ø 4 mm) is highly recommended to prevent corrosion in damp environment. Before drilling, physically hold the product against the wall at desired position and mark the hole locations. All the fixing points must be respected. After installation and before use, make sure that all fixing screws are tightened securely. DO NOT use the grab bar as a step. DO NOT sit on or hang from the grab bar.

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