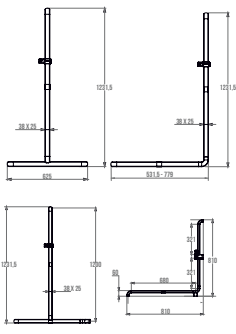
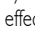
Poids maximum utilisateur / Max. user weight / Maximengewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht

Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes

<b>Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P :</b>		045285	045345
		045286	045346
		045295	045365
		045296	045335
<b>Fonction</b> <b>Function</b> <b>Funcție</b> <b>Función</b> <b>Funzione</b> <b>Função</b> <b>Funktion</b>	Permet à l'utilisateur de prendre appui en toute sécurité. Allows the user to lean safely. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite al usuario apoyarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers.		
<b>Descriptif</b> <b>Description</b> <b>Beschrijving</b> <b>Descripción</b> <b>Descrizione</b> <b>Descrição</b> <b>Beschreibung</b>			045285 120 kg 045286 120 kg 045295 120 kg 045296 120 kg 045345 120 kg 045346 120 kg 045365 110 kg 045335 110 kg 045285 2,250 kg 045286 2,650 kg 045295 2,450 kg 045296 2,750 kg 045345 2,450 kg 045346 3,150 kg 045365 3,700 kg 045335 2,050 kg
<b>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer</b>		A04500004	
<b>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum</b>		15/12/2023	
<b>Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr.</b>		00	
<b>Date / Date / Datum / Fecha / Data / Datum</b>		15/12/2023	

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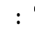
**GB**  **Do not install this equipment before reading the following instructions:**  
 **Fitting instructions.** Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected.  **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar, use it for assistance **ONLY.** **The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.**  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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