
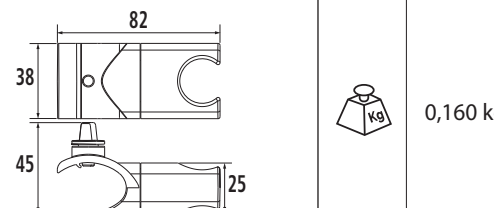
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	049314 049318
Fonction Function Functie	Permet de maintenir, orienter et régler la hauteur de la douche. Allows to maintain, direct and adjust the height of the bracket. Om de hoogte van de douchekop vast te zetten, te draaien en af te stellen
Función Funzione Função Funktion Назначение	Permite mantener, orientar y ajustar la altura del soporte de ducha. Permette di mantenere, orientare e regolare l'altezza del saliscendi. Permite manter, orientar e ajustar a altura do duche. Zur Halterung, Ausrichtung und Höheneinstellung der Handbrause. Держит душевую лейку и позволяет регулировать ее наклон и высоту
Descriptif Description Beschrijving Descripción Descrição Beschreibung Описание	 0,160 kg
Numéro notice / Instructions sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A04931401
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	02/05/2013
Révision n° / Revision No. / Revisiennr. / Revisión n° / Revisão n° / Revisionsnr. / № редакции	02
Date / Date / Datum / Fecha / Data / Datum / Дата	15/12/21

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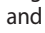GB ⚠ Do not install this equipment before reading the following instructions:  Fitting instructions. See drawings below.  Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. CAUTION: DONOT USE cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear

water. ⚠ Safety warnings. In order to be effective, the hand shower bracket must be installed according to the instructions. To guarantee the user's safety, you are advised not to hold on to or lean on this bracket. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this bracket.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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