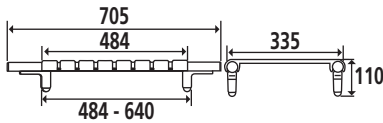
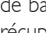
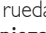
Référence G / Reference G / Referentie G / Referencia G / Codice G / Referència G / Referenz G / Код G :		134100 134106 134108
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :		047610
Fonction Function Functie Función Funzione Função Funktion Назначение	Permet de prendre une douche en position assise dans la baignoire Allows to take a shower in a seated position in the bathtub Zo kan u zittend een douche nemen in de badkuipen Permite tomar una ducha en posición sentada en la bañera Permette di prendere una doccia in posizione seduta nella vasca Permite tomar um duche sentado na banheira. Ermöglicht es, in der Badewanne im Sitzen zu duschen. Позволяет принимать душ сидя в ванне.	
Descriptif Description Beschrijving Descripción Descrição Beschreibung Описание		 140 kg  2,714 kg
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A13410026	
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	25/09/2002	
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	07	
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	15/12/2021	

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GB  **Do not install this equipment before reading the following instructions:**  **Fitting instructions.** For bathtubs internal width 484 to 640 mm. Proper fitting of the supports against the sides of the bathtub is essential for your safety. Once you have obtained the desired width, tighten the thumbwheels moderately to avoid damaging the thread.  **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The safety bath board adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The safety bath board is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this safety bath board.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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