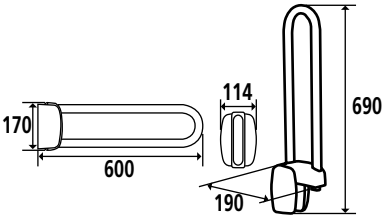
					
<p>Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux. This product is conform with the directive 93/42/EEC for medical products. Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur. Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos. Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medicali. Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte.</p>					
Référence G / Reference G / Referentie G / Código G / Codice G / Referència G / Referenz G :	488601				
Référence P / Reference P / Referentie P / Código P / Codice P / Referència P / Referenz P :	048860				
Fonction Function Functie Función Funzione Função Funktion	Permet à l'utilisateur de prendre appui en toute sécurité. Allows a safety leaning. Zo kan de gebruiker in alle veiligheid ondersteund worden Permite de apoiarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança Zum sicheren Abstützen des Benutzers				
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung	 <table border="1" style="float: right; margin-left: 20px;"> <tr> <td></td> <td>150 kg</td> </tr> <tr> <td></td> <td>1,977 kg</td> </tr> </table>		150 kg		1,977 kg
	150 kg				
	1,977 kg				
Numéro notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual / Gebrauchsanweisungsummer	A48860021				
Date de création / Creation's date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum	01/12/2002				
Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr.	10				
Date / Date / Datum / Fecha / Data / Data / Datum	24/04/2013				

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GB ⚠ Do not install this equipment before reading the following instructions:  **Fitting instructions.** Advised height for the upper bar: 80cm in toilets and showers, or at the height of the washbasin. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. On walls unable to support sufficient weight, there are 3 possible solutions: use of the vertical support post, the stand, or a reinforcement plate fitted behind the wall. Screws and plugs should be selected according to the type of wall. The 4 fixing points have to be necessarily respected.  **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. CAUTION: DO NOT USE cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠ Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user, use it for assistance ONLY. DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS
Distributeurs : GODONNIER SAS et PELLET ASC