Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя. Poids net / Net weight / Nettogewicht / Peso neto / Peso netto / Peso líquido / Nettogewicht / Вес нетто

<b>Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :</b>		049530	049550
		049580	049630
		049640	
<b>Fonction</b>	Permet à l'utilisateur de prendre appui en toute sécurité.		
<b>Function</b>	Allows the user to lean safely.		
<b>Functie</b>	Zo kan de gebruiker in alle veiligheid ondersteund worden.		
<b>Función</b>	Permite al usuario apoyarse con seguridad.		
<b>Funzione</b>	Permette di prendere appoggio con sicurezza.		
<b>Função</b>	Permite ao utilizador apoiar-se em toda a segurança.		
<b>Funktion</b>	Zum sicheren Abstützen des Benutzers.		
<b>Назначение</b>	Служит для пользователя надежной опорой.		
<b>Descriptif</b> <b>Description</b> <b>Beschrijving</b> <b>Descripción</b> <b>Descrizione</b> <b>Descrição</b> <b>Beschreibung</b> <b>Описание</b>			049530 - 049550 049630 - 049640
			150 kg
			049580
			120 kg
			049530
			0,480 kg
	049550		
	0,640 kg		
	049580		
	1,131 kg		
	049630		
	0,908 kg		
	049640		
	0,840 kg		
<b>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции</b>	A49530801		
<b>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания</b>	14/03/2011		
<b>Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции</b>	04		
<b>Date / Date / Datum / Fecha / Data / Data / Datum / Дата</b>	01/04/2021		

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P = PELLET ASC

**GB Do not install this equipment before reading the following instructions:**

**Fitting instructions.** Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected.

**Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

**Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar, use it for assistance ONLY. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.** ✂ At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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