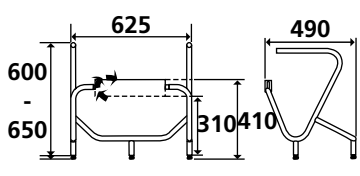
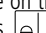
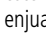
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :	047552 047553
Référence G / Reference G / Referentie G / Referencia G / Codice G / Referència G / Referenz G / Код G :	074190 074191
Fonction Fonction Functie Función Funzione Função Funktion Назначение	Permet de prendre appui pour se relever des toilettes Allows to take support to get up from toilet Zorgt voor steun om zich van het toilet op te heffen Permite apoyarse para levantarse del aseó Permette appoggiarsi per alzarsi nella toaletta Permite dar apoio para levantar-se da sanita. Zum Aufstützen beim Aufstehen von der Toilette Служит опорой при подъеме с унитаза
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание	 <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p>--</p> </div> <div style="text-align: center;">  <p>3,660 kg</p> </div> </div>
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	S00001444
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	01/12/2002
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	03
Date / Date / Datum / Fecha / Data / Datum / Дата	23/04/2013

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GB ⚠ Do not install this equipment before reading the following

instructions:  **Fitting instructions.** Put physically the product on the toilet. The mobile part of the frame has to be on the top of toilet. We recommend to fit the frame using the cover seat fixing points.  **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The safety frame adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. To guarantee the user's safety, it is not advised to rise up on the frame. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this safety frame.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS
Distributeurs : GODONNIER SAS et PELLET ASC