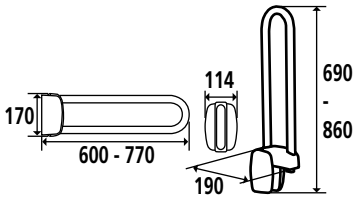
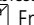
Référence P / Reference P / Referentie P / Código P / Codice P / Referència P / Referenz P :	048865 048880 048885
Fonction Function Functie Función Funzione Função Funktion	Permet à l'utilisateur de prendre appui en toute sécurité. Allows a safety leaning. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite de apoiarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers.
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung	 <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p>140 kg</p> </div> <div style="text-align: center;">  <p>2,500 kg</p> </div> </div>
Numéro notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual / Gebrauchsanweisungsnummer	S00001687
Date de création / Creation's date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum	13/11/2008
Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr.	02
Date / Date / Datum / Fecha / Data / Data / Datum	24/04/2013

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**GB ▲ Do not install this equipment before reading the following instructions:**  **Fitting instructions.** Advised height for the upper bar: 80 cm in toilets and showers, or at the height of the washbasin. Before drilling into the wall, physically

hold the product against the wall and mark the position of the holes to be drilled. On walls unable to support sufficient weight, there are 3 possible solutions: use of the vertical support post, the stand, or a reinforcement plate fitted behind the wall. Screws and plugs should be selected according to the type of wall. The 4 fixing points have to be necessarily respected.  **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **▲ Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user, use it for assistance ONLY. DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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**Fabricant : GODONNIER SAS**