Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	052210
Fonction Function Functie Función Funzione Função Funktion Назначение	Permet de régler la hauteur du lavabo Allows to adjust the height of the washbasin Hiermee wordt de hoogte van de wastafel afgesteld Permite regular la altura del lavabo Permette di regolare l'altezza del lavabo Permite de regular a altura do lavabo Sie dient zur WaschbeckenhöhenEinstellung Позволяет регулировать высоту раковины
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание	 ≤ 700 mm 28 kg
Número notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	S05221001
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	22/02/2012
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	04
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	21/07/2015

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Do not install this equipment before reading the following instructions:

Fitting instructions. This support is designed for any type of washbasin whose fastening distance lies between 180 and 370 mm. For the fitting, see drawings below.

Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

Safety warnings. The washbasin support functions correctly if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. To guarantee the user's safety, it is not advised to rise up on the washbasin. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this washbasin support.

At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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